

CheckUp

AT-HOME WELLNESS TEST FOR CATS

CheckUp provides a quick, simple and reliable testing method to observe the general wellness of cats through at-home urine testing.



This is ideal for those who want to monitor their cat's health in between visits to the clinic, or for those who require regular monitoring through urine collection as recommended by a veterinarian.

CheckUp can detect the most common issues among domestic cats including high glucose levels, kidney failure, urinary tract infection, and blood in the urine. By using CheckUp, you can detect a health condition in time to prevent health deterioration and improve your cat's lifespan.

Benefits

- Ideal for aging & overweight cats
- Easy to use
- 60 second testing
- Noninvasive and stress-free
- Biodegradable & recyclable
- On-the-spot test results for 4 conditions
- 100% natural and non-toxic
- Recommended by top veterinarians
- Obtain a clean, uncontaminated sample

IN THE BOX

Our patented hydrophobic litter has been used by thousands of veterinary clinics and hospitals in 44 countries worldwide since 2010.



2 lbs (900 g)
hydrophobic litter



2 x identical testing strips
*Repeat test with second strip if necessary



1 x sample collection pipette



1 x sample collection vial

Compliance from Cats

Sand is the natural environment for cats, allowing our product to yield quick compliance. The hydrophobic sand feels and smells just like regular sand, so cats are very cooperative when sample collection is required.

HOW TO USE



1 Pour the litter
Pour all hydrophobic litter in a clean and empty litter tray.



2 Wait for your cat
Allow your cat to urinate freely on the hydrophobic litter.




3 Collect samples
Easily collect urine from the top of the sand with the pipette.



4 Fill the vial
Fill the vial using the pipette to at least 3/4.



5 Dip the strip
Simply dip the strip into the vial until all squares are submerged.

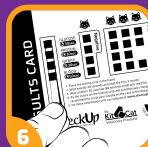

6 Read results
Quickly place the strip next to the Results card and follow instructions underneath the table.



7 Keep the sample
If necessary, close the vial with the cap and keep in the refrigerator.



8 Further analysis
If test shows any positive result, take the sample to your veterinarian for further analysis.

RESULTS CARD



Coastline Global
 After dipping the testing strip into the vial of collected urine, quickly place it into the frame above. Within 30 seconds, read the first 3 results and circle the matching colors. After another 30 seconds (60 seconds since the dipping), read the 4th result and circle the matching color.



CheckUp

AT-HOME WELLNESS TEST FOR DOGS

CheckUp provides a quick, simple and reliable testing method to observe the general wellness of dogs through at-home urine testing.



This is ideal for those who want to monitor their dog's health in between visits to the clinic, or, for those who require regular monitoring through urine collection, as recommended by a veterinarian.


CheckUp can detect the most common issues among domestic dogs including high glucose levels, kidney failure, urinary tract infection and blood in the urine. By using CheckUp, you can detect a health condition in time to prevent health deterioration and improve your dog's lifespan.

Benefits

- Ideal for aging & overweight cats
- Easy to use
- 60 second testing
- Noninvasive and stress-free
- Fast and Easy Assembly
- On-the-spot test results for 4 conditions
- 100% natural and non-toxic
- Recommended by top veterinarians
- Obtain a clean, uncontaminated sample

IN THE BOX

This innovative twist on the industry standard method for canine urine collection provides a simple to use and assemble telescopic pole and disposable cup, which makes collecting urine clean and comfortable.

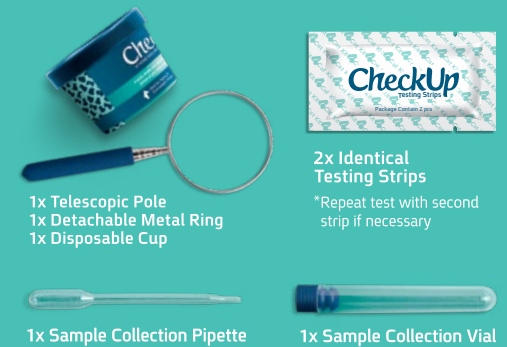

Compliance and Comfort

The telescopic pole extends to 60 cm, not including the length of the arm of the person conducting the test. This allows pet owners to maintain a comfortable distance from the dog while urinating, to avoid any mess or interference with the dog.

HOW TO USE

- Pole assembly**
Attach the metal ring to the telescopic pole by rotating it.
- Place the cup**
Pull the metal ring to extend the pole and place the cup in it.
- Collect sample**
Catch a sample while dog is urinating and bring home for testing.
- Fill the vial**
Fill the vial at least 3/4 way using the pipette.
- Dip the strip**
Simply dip the strip into the vial and make sure all squares are submerged.
- Read results**
Quickly place the strip next to the Results card and follow the instructions underneath the table.
- Keep the sample**
If necessary, close the vial with the cap and keep in the refrigerator.
- Further analysis**
If test shows any positive result, take the sample to your vet for further testing.

RESULTS CARD

Color	Glucose	Protein	pH	Blood
Light Blue	Negative	Negative	5.0 - 6.0	Negative
Light Green	Trace (+)	Trace (+)	6.5	Small (+)
Light Yellow	1+	1+	7.0	Medium (+)
Yellow	2+	2+	7.5	Large (+)
Orange	3+	3+	8.0	Large (+)
Dark Orange	4+	4+	8.5	Large (+)
Red	5+	5+	9.0	Large (+)
Dark Red	6+	6+	9.5	Large (+)

Coastline Global

